

The **Bio-logic External SEP Stimulator** option for the Navigator Pro AEP System supports Electroneuronography (**ENoG**)

Assists in the diagnosis & treatment of facial nerve paralysis


- Evaluation of facial nerve viability is critical in management of facial nerve disorders
- Provides an objective measure of facial nerve integrity
- CPT Code for Electroneuronography — 92516

ENoG test protocol

- Electrical stimulation of facial nerve near mastoid with measurement of motor response recorded near naso-labial fold
- Compares response amplitude of the abnormal side to that of the normal side

External SEP stimulator

- Navigator Pro system triggers SEP stimulator to deliver stimuli & records responses
- Provides constant current pulses of brief duration for transcutaneous stimulation
- Output current is continuously variable over range of 1 to 100 milliamps

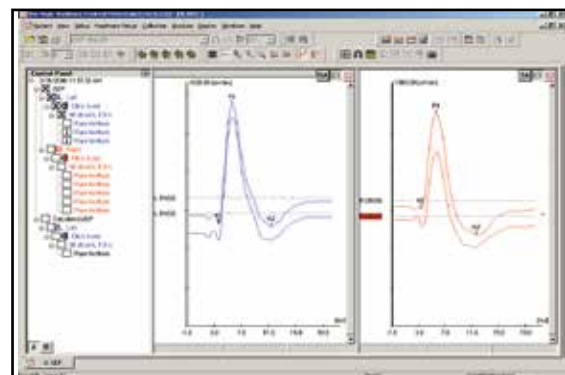

ENoG responses compared side-by-side on the Bio-logic Navigator Pro system

Add the ENoG package to a new or existing Navigator Pro system to expand your patient services



Bar electrode with handle is easy to place over the stimulation site

Trusted Reliability.
Flexible Innovation.
Proven Excellence.



Digitimer Stimulator



Digitimer SEP Stimulator



Ordering Information

Components connect the Navigator Pro to the Digitimer for electrical stimulation.

Electroneuronography

Item	Description	Quantity
SEP-KIT	Evoked Response Somatosensory Kit	1 each
Kit Includes		
580-STRB04	Interface Cable, Digitimer BNC to Nav Pro Trigger Cable	1 each
531-ADAP13	3.5 mm female - 6.35 mm male adapter/ ENoG Trigger Cable	1 each
101803	Reusable Bar electrode - 1 m lead	1 each
101898	T-Handle for Bar Electrode	1 each

For questions, please contact Natus customer service at 1-800-303-0306 or visit our web site at www.natus.com

Your Local Distributor:

