

## Olympic Papoose Board Accessories

### Head immobilizer:

Complete immobilization of head. Attaches quickly to board. Adjustable Velcro straps. Foam pads for comfort. When ordering, specify Papoose Board size.



### Flap extenders:

For immobilizing larger children. Two wide straps, with Velcro adjustment, attach to chest and abdominal flaps of Regular Size Papoose Board. Permits greater adjustment range.



### Arm immobilizer:

For Small and Regular Size Papoose Boards. Slides under board to firmly secure arms. Velcro arm straps adjust laterally.



All Olympic Papoose Boards are latex-free.  
Olympic Papoose Boards are fully guaranteed for one year.

### Ordering information

#### Olympic Papoose Boards

Small Size For infants up to 24 months  
Includes head & arm straps  
**CAT. NO. 50510**

Regular Size For children ages 2 to 6 years  
Includes head & arm straps  
**CAT. NO. 50500**

Large Size For children 6 to 12 years  
Includes head & arm straps  
**CAT. NO. 53600**

Extra-Large Size For teenagers & adults  
Includes head & arm straps  
**CAT. NO. 54610**

#### Head Immobilizers

For Small Size Papoose Board  
**CAT. NO. 50518**

For Regular Size Papoose Board  
**CAT. NO. 50508**

For Large Size Papoose Board  
**CAT. NO. 53608**

#### Other Accessories

Arm Immobilizers  
For Regular & Small Size Papoose Boards  
**CAT. NO. 50509**

Flap Extenders  
For Regular Size Papoose Board  
**CAT. NO. 50524**

### Information or Ordering

**Call 1-866-753-4823**

(toll-free in US/Canada)

**natus.**

Natus Medical Incorporated  
1501 Industrial Road  
San Carlos, CA 94070 USA  
1-800-303-0306  
1-650-802-0400  
[www.natus.com](http://www.natus.com)

© 2012 Natus Medical Incorporated. All Rights Reserved. All product names appearing on this document are trademarks or registered trademarks owned, licensed to, promoted or distributed by Natus Medical Incorporated, its subsidiaries or affiliates.

Safe and secure immobilization –  
Ages 3 months to teenagers and adults



Used in thousands of clinics, hospitals and dental offices

For safe immobilization of agitated patients



# Olympic Papoose Boards are indispensable aids in thousands of hospitals and in doctors' & dental offices

## Quick & easy to apply

- Three sets of opposing canvas flaps with Velcro® fasteners fold over the patient's body
- The diagonal shape of each board causes the flaps to criss-cross, providing a snug fit

## Easy to clean

- Simply wipe the board with liquid disinfectant or soap and water
- Flap sets and arm straps can be machine washed

## Selective immobilization permits exposure while maintaining control

- Any of the pairs of flaps may be opened while the child is still held securely by remaining flaps and/or head & arm straps



Place child on Papoose Board and secure arms with adjustable Velcro arm straps. Straps slide up or down to accommodate arm length.



Close abdominal flaps. Generous area of Velcro on flaps provide comfortable, positive immobilization. Patient is securely controlled.



Now close the leg flaps. The diagonal shape of the board causes the flaps to crisscross and fit snugly. Struggling is discouraged.



Finally, fasten shoulder flaps and head strap. Arms can be extended through the openings in the shoulder flaps if desired.

Hangs on wall For compact storage.

Infinite adjustment Generous areas of Velcro on flaps.

Detachable flaps Easily cleaned.

Rigid Board Lightweight, durable. Wipes clean.




Head Strap Adjusts up or down. Foam padded.

Arm Holes Permit exposure of one or both arms.


Arm Straps With secure Velcro closure. Adjust up or down.

Diagonal Shape Makes flaps conform to body shape.

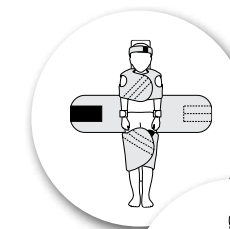


### Abdomen Exposed

Chest and leg flaps closed. Arms and head held by straps.

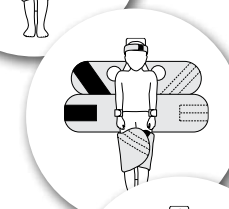


### Upper Body Exposed

Leg flaps closed. Arms immobilized by Velcro arm straps.

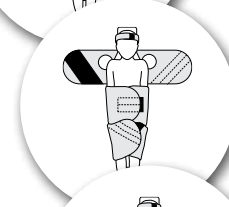


### Chest Exposed

Abdominal and leg flaps closed to securely hold patient.

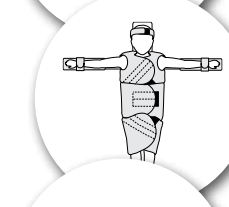


### Arms Exposed

Through opening in chest flaps. Can be held on arm immobilizer (optional).

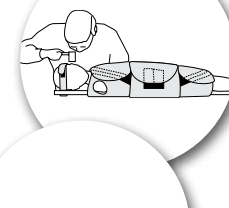

### Head Strap

Adjustable Velcro. Can move up or down. (Optional head immobilizer available.)

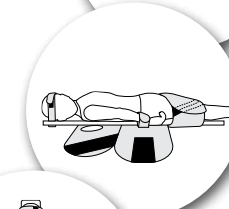


### Prone Position

Leg flaps closed. Arms held by straps. Head secured with head strap.



### Legs Exposed

Chest and abdominal flaps closed for body immobilization.

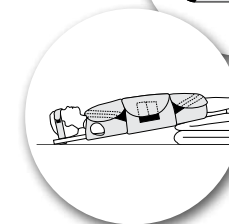

### Trendelenburg Position or Reverse Trendelenburg.