

When you only need a drop of blood, **SugarPlum** is the glucose lancet to use.




Wish you could draw just a single drop of blood for glucose tests? Now you can – with the SugarPlum glucose lancet.

- The same lancet style used for adult glucose tests – but sized for babies
- Two controlled depths:
 - 1.0mm depth for micropreemies to full-term babies
 - 1.8mm depth for full-term babies and adults
- Single-use needle retracts and locks after use
- Economical
- Sterile

SugarPlum and NeatNick– perfect companions for all your babies' blood draws!


Use SugarPlum for glucose tests that require just a drop of blood. Use our NeatNick sweeping blade lancet for PKU's, CBC's, and other tests. With NeatNick and SugarPlum you can choose the right lancet for all your babies' blood draws.

SugarPlum's patented design makes glucose draws quicker, easier and more comfortable for your babies. Press against the heel – SugarPlum automatically fires at the correct pressure. Before and after firing, the needle is safely concealed in the plastic body. 1.0mm depth for micropreemies to full-term babies (yellow); 1.8mm depth for full-term babies and adults (gray).

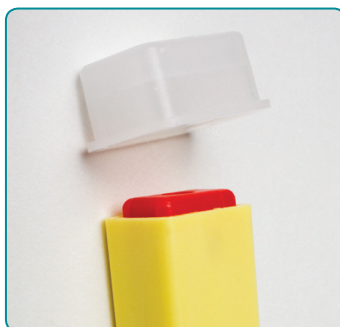

1. Sterile
2. Single-use needle retracts and locks after use
3. Two controlled depths of penetration:
 - 1.0mm – yellow
 - 1.8mm – gray

Using SugarPlum is safe and simple!

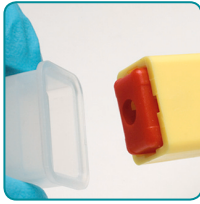

1. Remove protective cap



2. Place the raised red platform end of lancet on the site



3. Gently press down to activate the lancet

NeatNick Heel Lancets



Preemie Lime Green



Full-term Mango Orange

Ordering Information

SugarPlum Glucose Lancet and NeatNick Sweeping Action Heal Lancet are sterile, single-use disposable "sharps" used to puncture an infant's heel for collecting a blood sample and are **Latex-free, DEHP-free and BPA-free**. See instructions for use. For a product trial or questions, please contact Natus customer service at 1-800-303-0306.

SugarPlum Glucose Lancet

For micropreemies to full-term babies

| Item | Description | Quantity |
|---------|---|-------------|
| 1040043 | Un-bagged, 1.0mm depth (g26) Yellow | 100 per box |
| 1040044 | Individually bagged, 1.0mm depth (g26) Yellow | 100 per box |

For full-term babies and adults

| | | |
|---------|---|-------------|
| 1040041 | Un-bagged, 1.8mm depth (g23) Gray | 100 per box |
| 1040042 | Individually bagged, 1.8mm depth (g23) Gray | 100 per box |

NeatNick Sweeping Action Heel Lancet

| Item | Description | Quantity |
|---------|---|-------------|
| 1030083 | For babies weighing less than 1,800 - 2,000 grams Preemie size: Lime Green | 100 per box |
| 1030085 | For babies weighing more than 1,800 - 2,000 grams Full-term size: Mango Orange | 100 per box |

natus.

Natus Medical Incorporated
1501 Industrial Road
San Carlos, CA 94070 USA
1-800-303-0306
+1-650-802-0400
www.natus.com

© 2013 Natus Medical Incorporated. All Rights Reserved. All product names appearing on this document are trademarks or registered trademarks owned, licensed to, promoted or distributed by Natus Medical Incorporated, its subsidiaries or affiliates.